



Supply and Service of Medical Devices

www.trimbio.co.uk 01403 597 597

How to Clean a Zimmer Enpuls 2 Shockwave Unit.

It is very important to keep the shockwave applicator clean, internally and externally. The unit is supplied with protective caps to go over the treatment tips, if these are not used when the applicator is being applied to a patient and gel / lotion / cream / oil or any lubricate then the fluid will ingress the applicator and start to cause damage over a period of time. Please ensure that protective caps are always used when a fluid is used with device and patient.

Whichever way the applicator is used with or without caps it should be inspected regularly, this should be every day if it used daily, or once a week if only used a few times a week.

To inspect the applicator:

- 1, Make sure the device is turned off and not plugged into the mains supply.
- 2, Unplug the applicator / gun from the unit.
- 3, Remove the tip that is present on the applicator – this unscrews from the end.



4, Inspect the internal recess where the tip has been removed from, if the unit is new this should be a silver colour, and look clean. If the inside looks dark and dirty it needs cleaning.

5, To clean the applicator, use the small brush to clean the inside of the applicator, and the larger brush for the external part.






If required apply a small amount of Isopropyl Alcohol 99.9% to help remove any debris, this can be sprayed into the gun.



Then use the brushes again to help remove any debris, Please note the applicator must be drained out and cleaned off with a paper wipe and then left to dry off fully before the tip is replaced and the device is put back into use.



7, Inspect the tips, the device would have been supplied with 4, 1 x 6mm, 2 x 15mm and 1 x 20mm, the tip of these must not show signs of wear, once the end starts to roll over / flattened off then they should be replaced. If they are continued to be used then the applicator / gun can get damaged.

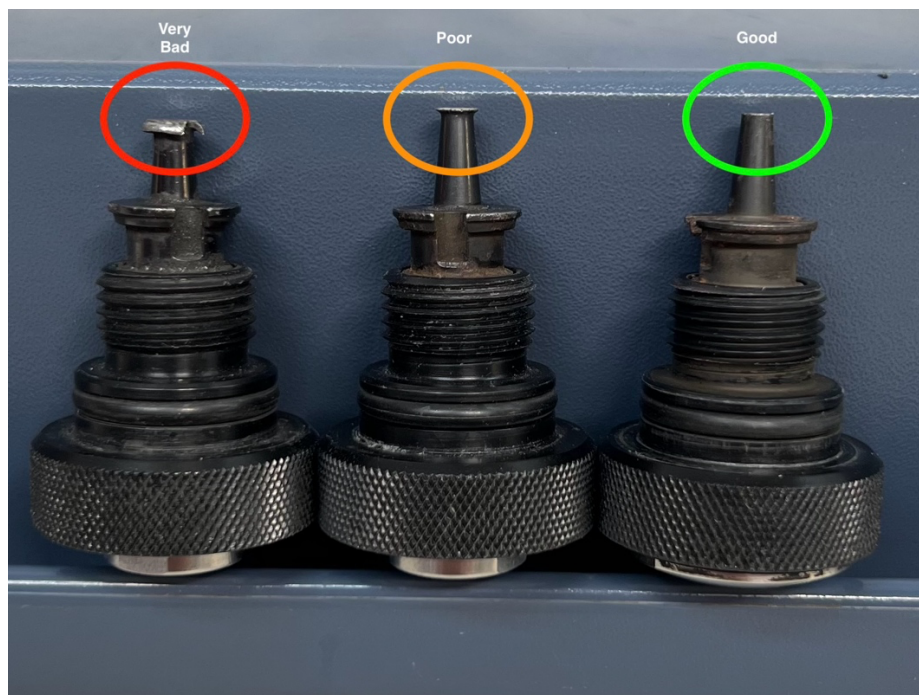

Cleaning Brushes & Fluid.

Order Codes:

CK002 – Brush 1 Small

CK003 – Brush 2 Large

CK004 – 500ml Isopropyl Alcohol in spray dispenser

Order from : [trimbio](https://www.trimbio.com)