

Cleaning Guidance -

Our Clinic Armour products are designed to be washed/wiped down with approved cleaning agents.

Cleaning Equipment required: soft, non abrasive cloth, cleaning agent in recommended concentrate. Either spray bottle or wipes are recommended. Dry towel

Prior to cleaning ensure the cover is fitted well with no loose areas. Firm fit not tight

1. Use an approved cleaning agent - over the entire cover (please find list of approved cleaning agents under the downloads tab)
2. Spread using a suitable cloth
3. Dry any excess cleaning agent off the surface with a dry towel to ensure a film residue from the cleaning agent doesn't build up. This can leave the surface "shiny"

This can be removed using a biodegradable detergent based cleaner.

The material has been tested against cleaning agents.

The normal cleaning instructions are to wipe the surface clean with a sponge or cloth with warm soapy water.

Clinic Armour covers can be washed in warm soapy water at up to 95°C.

Video on cleaning guidance - <https://www.youtube.com/watch?v=U4knwdyx3wU&t=2s>