

## Contents



x 8 releasable  
cable ties




x 1 T strap  
connector

## Instructions

**STEP 1:** Pass all straps under the examination table and through the eyelets. Ensure straps do not block the desired movement of any of your table sections.



**STEP 2:** Pull cable tie to tension, the straps should not be able to slide- you can trim the cable tie but allow for 5-10 mm of length to enable reuse.

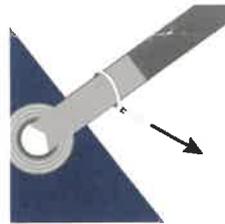

To release cable ties press the tab down and reverse the procedure above.



**STEP 3:** Thread the side strap nearest the foot end of the table through the T strap as shown and tighten. Next, feed the strap and end of the bed through the single loop and tighten. Use the cable ties as shown above for all fixing.

